



# GREEN&CLEAN M2

## Disinfection of the Suction System


GREEN&CLEAN M2



**1**  
Press dispenser pump twice  
(= 6 ml of concentrate)



**2**  
Fill container with cold water up to the indicator  
(= 600 ml of 1% dilution)

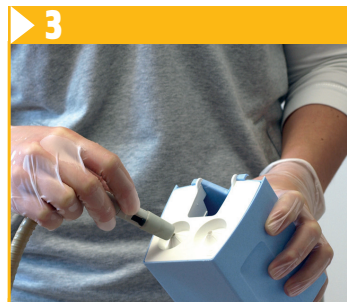

**3**  
Hold small suction tube on opening no. 1 until air is sucked in



**4**  
Hold large suction tube on opening no. 2 until air is sucked in



**5**  
Empty the remaining solution into the cuspidor

### Disinfection with GREEN&CLEAN M2:

- 2 x daily (noon/evening) and after surgical treatments
- After every treatment, rinse the suction system with cold water
- Change colour after 83 applications (1 bottle)
- Wear protective gloves when disinfecting
- Exactly follow dosage directions

### GREEN&CLEAN M2: Material friendly disinfection of the suction system

(including cuspidor, amalgam separator and air-water separation)

- dual system (red/green) active against resistance
- HBV/HIV/HCV, bactericide/fungicide (C. albicans), tuberculocide
- highly concentrated and easy to dose
- excellent deodorising and protein dissolving power
- actively defoaming
- DGHM/VAH listed, CE 0408